

# ENDO BAG<sup>®</sup>

## SPECIMEN RETRIEVAL BAG

A strong, coarse thread is provided with adequate length for quick grasping.

**Secure Closure:** Provides a reliable closure mechanism to securely contain specimens.

The memory ring maintains the opening of the bag.

The bag made of a **strong, tear-resistant** material, which provides extra safety for the surgeon.

**Endobag dispenser:** for simple insertion through a laparoscopic port.



**Super-thin and strong** bag material ensures safe specimen retrieval.



**Sterile and disposable** specimen retrieval bag.



**Easy insertion** through the trocar








**Cost Effective**

### Available Sizes

Ref No	Trocar Size	Mouth Dia(Inch)	Bag Length (Inch)	Volume (ml)
5010123	5mm	2.5"	4.0"	100ml
5010124	10mm	3.0"	5.0"	125ml
5010125	10mm	4.0"	6.0"	250ml
5010127	10mm	6.0"	7.0"	1000ml
5010128	15mm	6.0"	10.4"	1800ml

## Here's how to use the Morrisons Endobag step by step:

- 1. Open the Endobag:** Remove the Endobag from the sterile packaging, noting that it is preloaded in a dispenser.
- 2. Open the Button-hole Closure**
- 3. Insert the Endobag:** Introduce the Endobag with the dispenser through a 10mm trocar.
- 4. Grasp the Handle:** Hold the dispenser handle firmly from outside the body.
- 5. Deploy the Bag:** Use a non-toothed grasper to carefully pull the bag out from the dispenser inside the body cavity.
- 6. Remove the Dispenser:** Once the bag is deployed, remove the dispenser from the trocar.
- 7. Open, Place & Close:** Manually open the endobag within the body to prepare for specimen retrieval, place the specimen inside the bag, and securely close the endobag after the specimen is inside.
- 8. Lock the Bag:** Ensure the endobag is locked to safely retrieve the specimen.

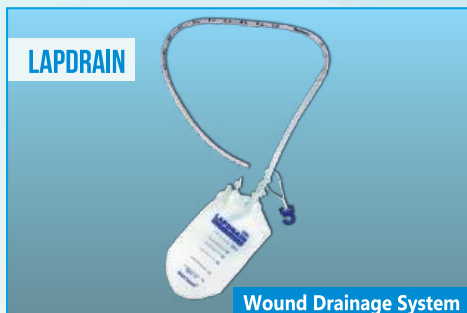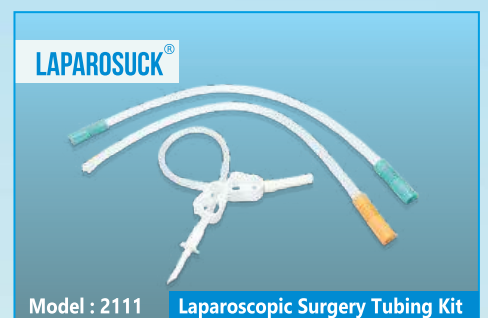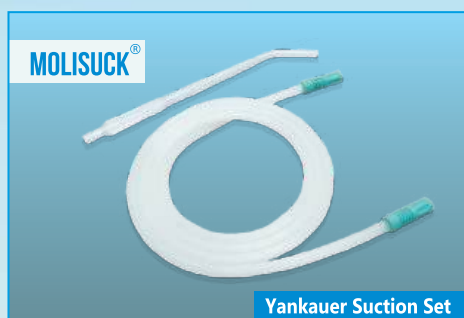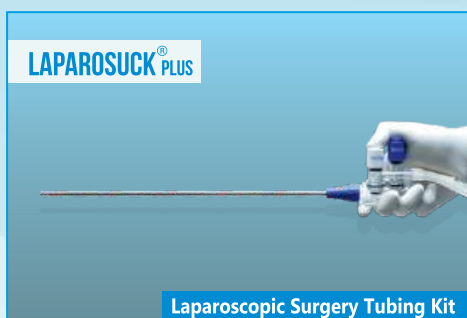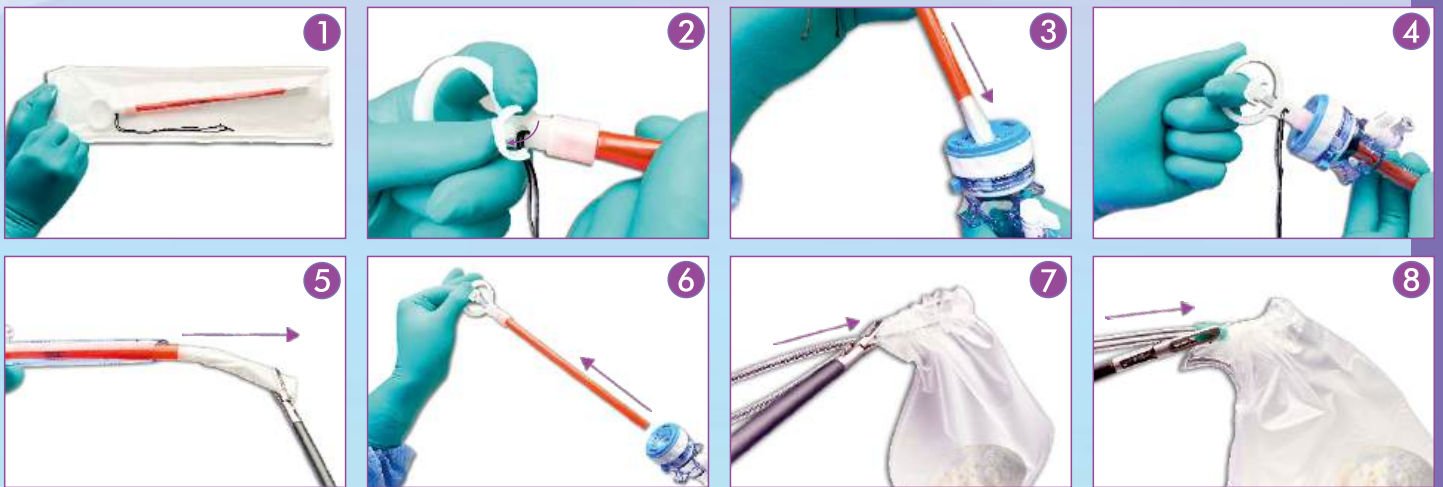

### Morrisons Lifecare Pvt. Ltd.

No.1 Mid Street, Krishnavakandigai, Mevalurkuppam,  
Sriperumbudur Taluk, Tamil Nadu, India - 602105.

Mob : +91-72000 90887, +91-96770 10128, Ph : 91-44-26155047

Email : [contact@morrisonslifecare.com](mailto:contact@morrisonslifecare.com), [sales@morrisonslifecare.com](mailto:sales@morrisonslifecare.com)

Web : [www.morrisonslifecare.com](http://www.morrisonslifecare.com)

